




BioSign® Flu A + B

An in vitro rapid qualitative test that detects influenza type A and type B antigens directly from nasal swab, nasopharyngeal swab and nasal aspirate/wash specimens*

SIMPLE PROCEDURE¹ FOR SWAB SPECIMENS

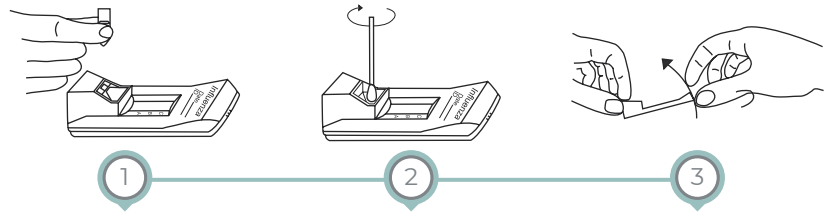

1 EXTRACT
Fill extraction well with reagent.

2 INCUBATE
Insert collection swab and incubate for 1 minute.

3 FLIP
Tip device for 1-2 seconds allowing liquid to flow.

READ RESULTS IN 10 TO 15 MINUTES.

READING RESULTS¹

VALID TEST RESULTS				INVALID TEST RESULTS		
NEGATIVE	POSITIVE	POSITIVE	POSITIVE	INVALID	INVALID	INVALID
	A LINE: Influenza type A	B LINE: Influenza type B	A & B LINE: Influenza type A & B	Repeat with new sample and device.		

PROVEN CLINICAL PERFORMANCE (BioSign® Flu A & B vs, PCR)

	INFLUENZA A	INFLUENZA B
POSITIVE % AGREEMENT	89.2% (95% CI:83.0-93.4%)	86.4% (95% CI:80.1-90.9%)
NEGATIVE % AGREEMENT	99.4% (95% CI:97.7-99.8%)	99.0% (95% CI:97.1-97.7%)

BioSign® Flu A + B, 25 tests/kit, REF #BSP-510-25

CPT CODE: Influenza A 87804QWReimbursement \$16.55²
Influenza B 87804QW-59.....Reimbursement\$16.55²

**THE TESTS YOU WANT.
THE ACCURACY YOU NEED!**

DISTRIBUTED BY:



**CLIA
WAIVED
FOR SWAB
SPECIMENS**



*Differentiates between
Influenza A and B*


INCLUDES:



**POSITIVE AND
NEGATIVE CONTROL
SWABS**



**INNOVATIVE FLIP
DESIGN WITH
ONBOARD SAMPLE
EXTRACTION**

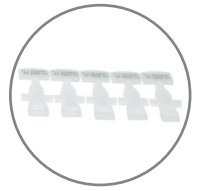

**PRE-MEASURED
DEVELOPER SOLUTION
CAPSULE** for increased
accuracy and ease of use



FLOCKED NASAL SWABS
for improved patient
comfort and superior
specimen collection



www.germainelabs.com



800.854.8446
210.692.4192
210.692.4198 Fax

©2019 Germaine Laboratories, Inc., CSM 0800-31 , Rev. 03-22
* CPT reimbursements vary, check your state for current amounts

¹ See Package Insert for complete details.
² Check with each state for exact reimbursement.
• Nasal Aspirate Specimens - Not CLIA WAIVED.