

AimTab™ REDUCING SUBSTANCES TABLETS

**FOR
FORENSIC
USE
ONLY**



AimTab™ Reducing Substances Tablets are for the semi-quantitative determination for the amount of reducing substances (generally glucose) in urine, stool, wine and other liquids.

**Quickly Test for
Glucose, Lactose,
Fructose, Galactose!**

**36 TABLETS
PER BOTTLE**



20 POUCHES PER BOX*
*1 Tablet per sealed foil pouch




TWO AimTab™ TABLET PACKAGING OPTIONS:

	SIZE	REF#
BOTTLED TABLETS	36 tests	13336
POUCHED TABLETS	20 tests	13320

RELATED PRODUCT

	SIZE	REF#
UA Liquid Controls	2x5mL	51116

- ⇒ Level 1 (1x5mL)
- ⇒ Level 2 (1x5mL)
- ⇒ Used for Glucose control
- ⇒ Store at 2° to 8°C
- ⇒ Includes hCG

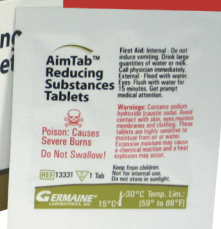

Same test procedure as Clinitest®, a registered trademark used by Bayer HealthCare LLC.

LOOK AT THESE GREAT FEATURES!

- ⇒ Quick & easy results for testing urine, stool, wine and other liquids.
- ⇒ Detects as little as 0.25 mg/dL of glucose.
- ⇒ Detects glucose and other reducing sugars such as lactose, fructose, galactose & pentoses.
- ⇒ Store at room temperature.



⇒ CAP / CLSI formatted procedures available via fax or email - just add your name!

5 DROP METHOD¹ (2 Drop Method also available)

- 1... With dropper in upright position, place **5 drops** of specimen in test tube. Rinse dropper with water and add 10 drops of water to test tube.
- 2... Drop **one tablet** into test tube. Watch while complete boiling reaction takes place. **DO NOT shake** test tube during boiling, or for the following 15 seconds after boiling has stopped.
- 3... At the **end of this 15-second waiting period**, shake test tube gently to mix contents. Compare color of liquid to Color Chart.
- 4... **Write down the percent (%) result** which appears on the color block that most closely matches the color of the liquid.



POISON: CAUSES SEVERE BURNS. DO NOT SWALLOW!

**THE TESTS YOU WANT.
THE ACCURACY YOU NEED!**

DISTRIBUTED BY:

www.germainelabs.com



800.854.8446
210.692.4192
210.692.4198 Fax

