




# BioSign<sup>®</sup> Flu A + B

An in vitro rapid qualitative test that detects influenza type A and type B antigens directly from nasal swab, nasopharyngeal swab and nasal aspirate/wash specimens\*

## SIMPLE PROCEDURE<sup>1</sup> FOR SWAB SPECIMENS

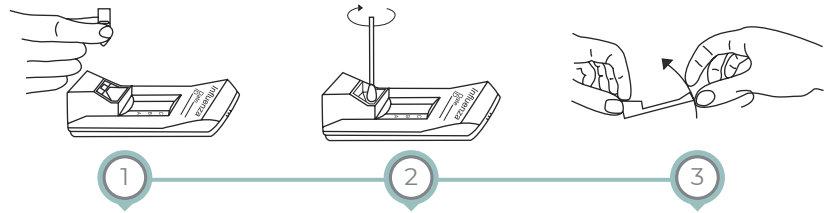

**1 EXTRACT**  
Fill extraction well with reagent.

**2 INCUBATE**  
Insert collection swab and incubate for 1 minute.

**3 FLIP**  
Tip device for 1-2 seconds allowing liquid to flow.

**READ RESULTS IN 10 TO 15 MINUTES.**

## READING RESULTS<sup>1</sup>

VALID TEST RESULTS				INVALID TEST RESULTS		
NEGATIVE	POSITIVE	POSITIVE	POSITIVE	INVALID	INVALID	INVALID
	A LINE: Influenza type A	B LINE: Influenza type A	A & B LINE: Influenza type B	Repeat with new sample and device.		

## PROVEN CLINICAL PERFORMANCE (BioSign<sup>®</sup> Flu A & B vs, PCR)

	INFLUENZA A	INFLUENZA B
<b>POSITIVE % AGREEMENT</b>	89.2% (95% CI:83.0-93.4%)	86.4% (95% CI:80.1-90.9%)
<b>NEGATIVE % AGREEMENT</b>	99.4% (95% CI:97.7-99.8%)	99.0% (95% CI:97.1-97.7%)

BioSign<sup>®</sup> Flu A + B, 25 tests/kit, REF #BSP-510-25

CPT CODE: Influenza A 87804QW .....Reimbursement \$16.55<sup>2</sup>  
 Influenza B 87804QW-59.....Reimbursement\$16.55<sup>2</sup>

**THE TESTS YOU WANT.  
THE ACCURACY YOU NEED!**

**DISTRIBUTED BY:**



**CLIA  
WAIVED  
FOR SWAB  
SPECIMENS**

*Differentiates between  
Influenza A and B*

**INCLUDES:**



**POSITIVE AND  
NEGATIVE CONTROL  
SWABS**



**INNOVATIVE FLIP  
DESIGN WITH  
ONBOARD SAMPLE  
EXTRACTION**



**PRE-MEASURED  
DEVELOPER SOLUTION  
CAPSULE** for increased  
accuracy and ease of use



**FLOCKED NASAL SWABS**  
for improved patient  
comfort and superior  
specimen collection



[www.germainelabs.com](http://www.germainelabs.com)



800.854.8446  
210.692.4192  
210.692.4198 Fax

<sup>1</sup> See Package Insert for complete details.  
<sup>2</sup> Check with each state for exact reimbursement.  
 • Nasal Aspirate Specimens - Not CLIA WAIVED.