



# ComplianceGold®

## Fecal Occult Blood Test - ER

PATIENTS RETURN MORE TESTS WITH COMPLIANCE! \*\*

CLIA  
WAIVED

ROOM  
TEMPERATURE  
STORAGE



A rapid, convenient, and non-offensive qualitative method for detecting occult blood in stool. Recommended for routine testing and mass screening programs for colorectal cancer detection.

**Compliance  
Saves Lives!™**

**Compliance Gold® FOB makes testing for occult blood convenient for patients.**

### \*\* COMPLIANCE MONITORING SYSTEM



### COMPLIANCE MONITORING\*\*

Our team provides a HIPAA Compliant program that reminds patients to return Collection Kits.



#### STEP 1

Patient reads HIPAA Compliant Patient Agreement Form, signs form and enters their phone number.



#### STEP 2

Doctor/physician faxes the Patient Agreement Form to the secure fax line at 888-797-4362.



#### STEP 3

A series of reminder messages will remind patients to return completed tests to their doctor.

### Look at these great features:

- ⇒ **Postage Provided for Return Mailers**  
Patient Friendly! No more trips to the post office to buy postage.
- ⇒ **Return Labels Included**  
Ensures tests are returned to the proper address.
- ⇒ **All Components Pre-Packaged in Mailer**  
Everything your Patients need in one Return Mailer.
- ⇒ **Reliability**  
Detects Hb at 0.6mg, on-slide controls.
- ⇒ **Enhanced Readability**  
Results are easy to read.

**Test results within 30 seconds.** Meets American Cancer Society multiple-day testing recommendation.



CPT CODE: G0394\* Colorectal cancer screening, 1-3 simultaneous determinations, guaiac. 82270/82272 - FOB by guaiac, patient given 3 slides for consecutive collection.



[www.germainelabs.com](http://www.germainelabs.com)



800.854.8446  
210.692.4192  
210.692.4198 Fax



**THE TESTS YOU WANT.  
THE ACCURACY YOU NEED!**

**DISTRIBUTED BY:**

# ComplianceGold®

## Fecal Occult Blood Test - ER

**Compliance Saves Lives!™**

### INSTRUCTIONS

CLIA  
WAIVED



#### PATIENT



**1.** Lift toilet seat and position several layers of tissue onto water in bowl. Deposit bowel movement onto tissue paper.



**2.** Use supplied applicator stick to obtain stool specimen.



**3.** Place test slide on a flat surface. Open flap of test slide. Open first flap of triple slide.



**4.** With applicator, apply thin smear of stool inside area I. Using same applicator, repeat from a different portion of the stool and fill area II.




**5.** Discard applicator, close tab and bend first slide. Separate and place into foil envelope. Do not seal envelope. Repeat procedure for next 2 bowel movements. Return to healthcare provider immediately after preparing last test.

#### PHYSICIAN




**1.** On back of slide, open perforated section, marked 1 and 2.

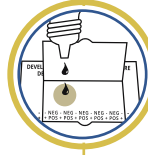

**2.** Apply two drops of Compliance Gold® FOB Developing Solution to exposed test paper.



**3.** Read results between 30 - 60 seconds. Any trace of blue is positive for occult blood. No indication of blue is negative.

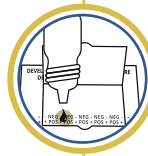

**4.** Add 1 drop of developer directly onto control area between positive (+) and negative (-) performance standards.



**5.** Read results within 30 seconds. Blue color should appear within 30 seconds on "POS (+)", no change should appear on "NEG (-)".

**TRI-SLIDES:  
50 TESTS PER KIT**



**SINGLES:  
100 TESTS PER KIT**

#### GUAIAEC FECAL OCCULT BLOOD TEST

##### FULL VERSION

The following kit includes Compliance Monitoring and Postage.

PRODUCT	REF#
Compliance Gold® FOB Test-ER, TRI-SLIDES	74252

##### ECONOMY VERSION

The following kits DO NOT include Compliance Monitoring and Postage.

PRODUCT	REF#
Compliance Gold® FOB Test-ER, TRI-SLIDES	74250
Compliance Gold® FOB Test-ER, SINGLES	74200

##### DEVELOPER

PRODUCT	SIZE	REF#
Compliance Gold® FOB Developer	8 x 10 mL	74208

**Our fecal occult blood products** are manufactured to the highest quality standards utilizing current Good Manufacturing Practices (cGMP) as overseen by the FDA and the International Standardization Organization (ISO) 13485:2016. These certifications attest to the quality standards set and achieved for the manufacture of our fecal occult blood products. Certified training is available for all registered visitors at: [www.compliancegold.org](http://www.compliancegold.org).

**THE TESTS YOU WANT. THE ACCURACY YOU NEED!**



[www.germainelabs.com](http://www.germainelabs.com)

